

Opacal

High pH value
for temporary
root canal
dressing



Opacal is a radiopaque, creamy and ready-to-use calcium hydroxide paste with a pH above 12. Paste delivery in root canal is ensured by dedicated application tip.



SMART PRODUCTS
FOR ENDO LOVERS


Opacal

Designed for temporary root canal dressing with high pH value (>12), high radiopacity (> 3 mm alu) and creamy consistency. Its aqueous composition and the related rapid ionic release *R.G. Fava & W.P. Saunders – Calcium hydroxide pastes: classification and clinical indications*) delivers performance in short treatment time procedures. The ready-to-use paste consistency and fine application tip offer a comfortable paste delivery and reduces the need for Lentulo instruments.


Composition Calcium Hydroxide 30%, Barium Sulfate 24%, excipient

Key benefits


- high pH value
- high radiopacity
- creamy consistency
- easy and comfortable paste delivery
- easy to remove (non-setting material)
- rapid ionic (aqueous composition)

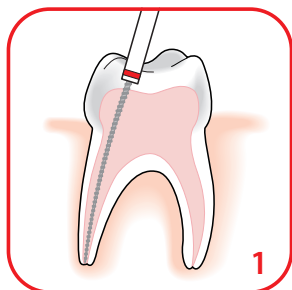

1 Prepare, clean and dry Meticulously the root canal.

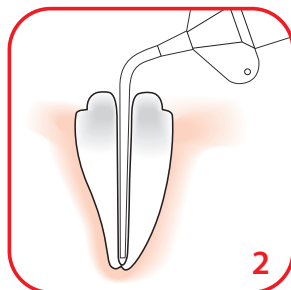

2 Fill the canal with Opacal from apical portion of root outward. Use the enclosed tip and stay at least 2 mm shy of apex.

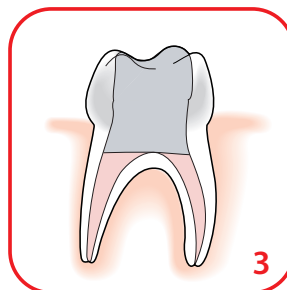

3 Fill the cavity with a temporary filling material for a tight sealing during intercession.



Opacal Calcium Hydroxide Paste		REF 11020
1	syringe of 3 ml	
10	single use application tips	
1	finger grip	

Opacal refill		REF 11022
10	single use application tips	



Produits Dentaires SA
pd-dental.com